Riboflavin (0.1%)

Description

RICROLIN®+ is a hypotonic ophthalmic solution containing Riboflavin (0.1%), specifically formulated to allow quick passage of Riboflavin into the corneal stroma, either through the healthy epithelium or after de-epithelisation or even by using a low intensity electrical field applied topically (iontophoresis). This solution, used on the eye, combined with an emitter of ultraviolet (UV-A) rays, is indicated for the conservative parasurgical treatment of the keratoconus and of corneal ectasia diseases. The aim of **RICROLIN®+** is also to provide a protective barrier to the cornea in order to attenuate the penetration of UV-A rays beyond the stroma, thus preserving the delicate internal structures of the eye (corneal endothelium, lens and retina) from the damage which radiation would cause to them. The Riboflavin solution of **RICROLIN®+** can actually reduce the intensity of the UV energy reaching the deep layers of the cornea by up to 95%. When administering **RICROLIN®+** using iontophoresis, the transepithelial riboflavin penetration, aided by the application of a low intensity electric field, facilitates the corneal cross-linking operation by shortening absorption time and thus reducing patient discomfort.

Indications

RICROLIN®+ in combination with the medical device emitter of UV-A for ophthalmic use, for corneal cross-linking (CXL) in eyes with primary corneal ectasias (evolving keratoconus, pellucid marginal degeneration) or iatrogenesis.

In particular, the treatment is indicated:

- in all ectatic diseases of the cornea, whether genetic or iatrogenic;
- in progressive keratoconus (as clinically and instrumentally documented).

Treatment (UV-A + Riboflavin) causes a delay in the progression of keratoconus.

The treatment prevents the need for corneal transplantation.

The treatment does not exclude the possibility of keratoplasty at a later stage.

Directions for use

For Cross-linking with de-epithelisation (CXL standard) (Epi-off)

starting from 15 minutes before UV-A irradiation, instil on the cornea one drop of **RICROLIN®+** every 1-2 minutes (at least 10 drops in 15 minutes)

For transepithelial Cross-linking (Epi-on)

starting from 30 minutes prior to UV-A irradiation, instil one drop on the cornea **RICROLIN®+** every 2 minutes (at least 16 drops in 30 minutes).

If you are using a corneal ring, leave the cornea covered by **RICROLIN®+** for 20-25 minutes before irradiation.

For Cross-linking using iontophoresis

Use by iontophoresis reduces absorption time to five minutes.

In the absorption stage, instil **RICROLIN®+** in the specific electrode applied on the eye, until the grid is completely covered. Then set the electric field generator and keep it running until the instrument signals completion of the absorption stage. When the absorption stage has been completed, remove the electrode and proceed with the UV-A irradiation according to the specific treatment protocol.

Contraindications

The product is contraindicated for known hypersensitivity to own components or other closely related substances from a chemical standpoint.

Side effects

There are no known side effects of a systemic nature or effects caused to the eye surface structures.

Ingredients

Riboflavin sodium phosphate, Sodium Edetate, Tromethamine, Sodium phosphate monobasic dihydrate, Sodium phosphate dibasic dihydrate, purified water.

Warnings

For irradiation parameters, refer to the instrument manual.

The product must be used by an ophthalmologist or under his or her direct control.

Use the product according to its directions for use.

Precautions

- Store away from light.
- In the hours before instillation, and until instillation begins, the product should be refrigerated at a temperature between +4° and +8°C.
- Do not use the product after one hour has passed from the time of its suction from the bottle.
- Any one bottle should be used for the same patient and for a single treatment.
- Do not use if the package is damaged.
- Do not use the product after the expiry date shown on the package.
- Do not store at temperatures above 25°C.
- This product is for topical ophthalmic use only.

Characteristics of medical devices with which **RICROLIN®+** can be used for iontophoresis

UV-A emitter: **VEGA®** or equivalent

Corneal electrode: **IONTOFOR CXL®**

Current generator: **I-ON CXL®**

Sterile syringe with Luer Lock connection

Packaging

1 bottle containing 1,5 ml of the solution

Date of issue of the latest version of
directions for use: ver. 3 dated 03/08/2016

SOOFT italia S.p.A.
C.da Molino, 17
Montegiorgio (FM) Italy
Tel. + 39 0734964096
www.sooft.it

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