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CE 0459

Treats and soothes eyelids affected by pathology

Cream containing 0.5 % hyaluronic acid (Sodium hyaluronate) Ophthalmologically and dermatologically tested 0 % preservatives. 0 % parabens. 0 % perfume. 0 % alcohol. 0 % lanolin. Epifree® Technology is an exclusive and patented manufacturing process. It guarantees the absence of all preservatives and maintains the integrity of the active ingredients.

PRESENTATION: 30 ml bottle with airless pump. Box of 5 single-dose units (sample).

COMPOSITION: Aqua (water), Caprylic/capric triglyceride, Tribehenin PEG-20 esters, Glycerin, Trisodium citrate, Xanthan gum, Sodium Hyaluronate, Taurine, Allantoin, Helianthus Annuus (Sunflower Seed oil) Retinyl Palmitate, Citric acid, R-alpha Lipoic acid sodium.

WHEN SHOULD ILAST® CARE BE USED? Cream designed for treating and soothing eyelids affected by pathology: dryness and/or irritation (allergy, blepharitis) or delicate skin (after surgery). Ilast® Care helps to relieve skin irritation and dryness. Sodium hyaluronate moisturises and helps to repair the skin. In addition, Vitamin A possesses photo-protective qualities. Lipoic acid has antioxidant properties that counteract oxidative stress. It also has patented osmoprotective properties that help keep cellular water content in balance.

INSTRUCTIONS FOR USING ILAST® CARE - Cutaneous route, apply thinly and evenly to the eye contour area morning and evening in light circular massaging movements.

Bottle: - When using it for the first time, press the top of the dispenser 5 to 10 times in order to prime the pump. - Preferably, press on the top of the dispenser and take some cream on your index finger before application. - Be sure to wipe the tip of the dispenser after use (risk of bacterial contamination).

Single-dose unit: - Take a strip and detach one of the single-dose units. - Twist off the tip of the single-dose unit being careful not to touch the end once it has been opened. - Squeeze the single-dose unit to make the product come out. - Dispose of the single-dose unit after use.

WARNINGS: • Do not swallow. • Keep out of the reach of children. • Do not use after the expiry date marked on the product (risk of loss of performance and microbial contamination). • Do not use if the system shows wear: damage to the bottle, pump disconnected, etc. (risk of bacterial contamination). • Do not open the bottle (risk of bacterial contamination). • Use within 3 months of opening of the flask. • Store at a temperature between 2 and 25° C.

CONTRAINDICATIONS: Allergy to one of the ingredients.

WHAT IS EPIFREE TECHNOLOGY®? HORUS PHARMA laboratories have developed Epifree® Technology. This is an exclusive and patented manufacturing process, which guarantees the absence of preservative agents, while maintaining the integrity of the active ingredients. Thus, neither hyaluronic acid, which tolerates standard decontamination processes poorly, nor the other active ingredients are affected. The eyelids are a particularly sensitive and vulnerable part of the body. Closeness to the eye precludes the use of common skin treatments. As a result of the progress made by Epifree® Technology, the latest generation from the ILAST® product range is particularly suitable for cleansing and caring for skin that is highly sensitive, allergic or affected by a medical disorder.

Medical device complies with the requirements of directive 93/42/EEC. Date of brand application: (11/2011; rev 2017) EAN: 3401096077490

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