FR

ODM5®

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CE0459

- When using for the first time, check that the tamper-proof ring is intact, then break the seal.
- Remove the protective cap by pulling upwards.
- Apply 1 or 2 drops to the eye.
- Replace the protective cap after every use, ensuring the tip is free from any residual drops; if present, shake any off with a sharp wrist movement.

WARNINGS:

- Do not swallow.
- Do not use if bottle is damaged (risk of contamination).
- Do not use after expiry date printed on the bottle.
- Use within 3 months of opening (risk of contamination and loss of product performance).
- Store at room temperature (between + 2 and 25°C).
- Carefully close bottle after each use, ensuring the tip is free from any residual drops (risk of contamination).
- Avoid touching your hands, your eye or your eyelid with the tip of the bottle (risk of microbial contamination).
- Avoid touching or drying the tip with fingers or a non-sterile tissue (risk of microbial contamination).
- If your doctor has prescribed additional eye drops, wait at least 15 minutes between applications.
- Keep out of reach and sight of young children.

CONTRAINDICATIONS:

- Hypersensitivity to one of the components.
- Wearers of contact lenses (ODM5° is not compatible with contact lenses).

ADVERSE EFFECTS:

- In some cases transient disorders such as a slight sensation of burning or smarting and momentary blurring of the vision may occur during instillation.
- If there is a persistent adverse effect or one not described in this leaflet, stop using the product and consult your doctor.

Medical device compliant with the requirements of Directive 93/42/EEC amended.

Date marking affixed: october 2013.

Date of last revision of the text: December 2015.

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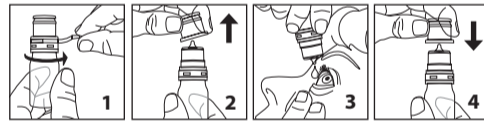
EN

ODM5®

Preservative-free hyperosmolar ophthalmic solution

PRESENTATION:

Preservative-free 10 ml multi-dose bottle.

COMPOSITION:

Sodium chloride 5%, Sodium hyaluronate 0.15%.
Other ingredients: Purified water, trisodium citrate dihydrate, citric acid monohydrate, hydroxypropylmethylcellulose.

ODM 5° contains sodium hyaluronate. This component is naturally present in the eye, and its lubricating properties protect the cornea.

WHEN SHOULD ODM 5° BE USED?

ODM 5° is recommended for the reduction of corneal oedema: oedema caused by corneal dystrophies, post-traumatic oedema and post-surgical oedema. Corneal oedema is characterised by an excess of fluid in the cornea. ODM 5°, a hypersaline solution attracts this excess fluid away from the cornea by an osmotic effect, enabling the cornea to regain a normal state of hydration for a brief time. In this way ODM 5° participates in improving your quality of vision.

HOW SHOULD ODM 5° BE USED?

- Wash your hands carefully before instilling the product.
- Instil 1 to 2 drops 4 times a day as recommended by your doctor. Apply the drops into the lower conjunctival sac of the eye, looking upwards and pulling the lower lid gently downwards.
- After instillation, keep the eye closed for 1 to 2 minutes.
- Avoid contact between the tip of the bottle and the eye or eyelids.
- Does not require rinsing after use.
- OCULAR ROUTE.**

Utiliser avant (année/mois)
Use before (year/month)
Zu verwenden vor (Jahr/Monat)
Porabite najkasneje do (leto/mesec)
Utilizar antes de (año/mes)
Te gebruiken tot (jaar/maand)

Ne pas utiliser si le conditionnement est endommagé
Do not use if package is damaged
Nicht verwenden, wenn die Verpackung beschädigt ist
Ne uporabite, če je embalaža poškodovana
No utilizar si el envase está dañado
Niet gebruiken indien de verpakking beschadigd is

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