EN

# ODM 5<sup>®</sup>

## Eye Ointment

### COMPOSITION

Sodium chloride 5% (Hypertonicity agent).  
Others ingredients: Purified Water, Wool fat, Paraffin white soft, Paraffin liquid.

### INDICATIONS

Temporary relief of corneal edema.  
The product works by drawing fluid out of the cornea to reduce swelling and discomfort associated with edema.

### INSTRUCTIONS FOR USE

Pull down the lower lid of the affected eye and apply a small amount of product (a little strip of 1 cm) to the inside of the eyelid every 3 to 4 hours, or as directed by a doctor.  
Avoid touching the eye with the dispenser and thoroughly clean the nozzle before closing. If you are wearing contact lenses, remove them before using eye ointment.  
The product is preservative free and does not contain Thimerosal or Benzalkonium Chloride.

### WARNINGS

Before use check the protective ring is machine sealed. Sterile until the first opening, do not use in case the tube is damaged. Close the tube properly after use and keep tightly closed. Consult your doctor if you are using other drugs or medical devices for ophthalmic use. The use of the product may induce burning and temporary irritation. In case the problem persists or in case of irritation, please consult your doctor. It is important to inform your doctor or pharmacist of any undesired effects not reported in the enclosed leaflet. Store away from sources of heat. Protect from freezing. Keep out of reach and sight of children. Discard the eye ointment 28 days after first opening. Do not use after the expiry date shown on the package. Remove contact lenses before use.

### CONTRAINDICATIONS

Do not apply the product in the case of ocular injuries.  
Do not use the product in the case of individual hypersensitivity towards the components.

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Velit Biopharma S.r.l.,