# Lingettes • Wipes

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CE 0459

# Lingettes • Wipes

**CLEANSING WIPES FOR SENSITIVE  
OR ABNORMAL EYELIDS AND SKIN**

- Gently cleanse the eyelids and lashes
- Remove residues, scales, crusts, lipids and dirt
- The protective action of hyaluronic acid (0.2%) and allantoin

From the ophthalmological expertise of Laboratoires Horus Pharma, Ilast® Wipes are the first cleansing wipes specifically formulated to be preservative, paraben, phenoxyethanol, alcohol, fragrance, colouring, lanolin and essential oil free to protect the eye and the eyelid.

Ilast® cleansing wipes mechanically remove scales, seborrhoeic residues, secretions and crusted matter present on the eyelids and along the roots of lashes due to eye conditions. Thanks to their formula and cleansing action, Ilast® cleansing wipes limit the loss of skin moisture, facilitate the healing process and prevent germ growth, which can cause secondary infections.

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Box of 20 pre-moistened sterile cleansing wipes ready for use. Preservative and alcohol free.

#### COMPOSITION

WATER, GLYCERINE, POLYSORBATE 20, TRISODIUM CITRATE, ALLANTOIN, SODIUM HYALURONATE, CITRIC ACID

#### INDICATIONS

Ilast® Wipes are recommended for cleansing diseased eyelids, especially the eyelid margin, in all forms of blepharitis and meibomian gland dysfunction and may be used before or after eyelid, eye or periocular surgery.

#### DIRECTIONS FOR USE

Steps to be taken twice a day, morning and evening. Do not rinse.

- 1 After washing and drying your hands, open the sachet and take out the wipe
- 2 Roll the wipe around your index finger
- 3 With your eye closed, gently rub the eyelid and lashes with the wipe using a circular movement
- 4 With your eye open, clean the free eyelid margin at the base of the lashes pulling the eyelid down slightly
- 5 Take another wipe and repeat the operation on the other eye

#### SPECIAL WARNINGS

For cutaneous use only, do not apply directly into the eyes.

Do not swallow.

Do not re-use (risk of contamination and loss of efficacy).

Do not use if the sachet is damaged.

For single use.

Keep out of reach of children.

Do not use after the expiry date shown on the sachet.

Store at a temperature between 2°C and 25°C.

#### CONTRAINDICATIONS

Hypersensitivity to one of the components.

Class IIa medical device, in accordance with the requirements of Directive 93/42/EEC.

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